

Wheelchair Oxygen Bracket

Product Code: RAWC106

- The Wheelchair Oxygen Bracket will hold an oxygen cylinder up to 12cm in diameter.
- It is finished in a chrome steel construction and can be attached easily to a wheelchair.
- The bracket will fit any wheelchair where the rear step tube is 22mm in diameter.
- Designed to fit the left hand side only of a wheel chair (left hand side when standing at the rear of the chair)



To Fit the Oxygen Bracket



Remove the step tube end cap.



Slide the Oxygen Bracket over the step tube.



Clamp the bracket around the vertical frame and secure into place with the bolt



Pharmaquip Healthcare - 1/7 Monte Khoury Drive Loganholme Queensland Australia 4129
PH:(07) 3806 4274 - www.pquip.com.au - sales@pquip.com.au