

# Wheelchair IV Pole

Product Code: RAWC107



- The Wheelchair IV Pole has two hooks and fits onto the rear step tube of a wheelchair
- The pole is then secured to the frame by a bracket.
- The IV Pole is made of chrome-plated steel and is height adjustable from 86cm to 156cm.
- Will fit any wheelchair where the rear step tube is 22mm in diameter.

## To Fit the IV Pole



Remove the step tube end cap.



Slide the IV Pole over the step tube.



Remove the screw that secures the backrest to the frame, then place the screw through the bracket and re-attach.



Pharmaquip Healthcare - 1/7 Monte Khoury Drive Loganholme Queensland Australia 4129  
PH:(07) 3806 4274 - [www.pquip.com.au](http://www.pquip.com.au) - [sales@pquip.com.au](mailto:sales@pquip.com.au)