

Transfer Bath Bench

Product Code: RBN102

Transfer Bath Benches are designed to assist patients who have difficulty transferring in and out of the bathtub. The height adjustable frame is made of a lightweight aluminium and the plastic seat has a non-slip textured surface. Drainage holes in the seat prevent the build up of water. The backrest is reversible to allow for access from either the left or right hand side.

Features

- Height adjustable
- Armrest provides extra security
- Wide base for extra stability
- Anodised aluminium frame tubing
- Suction cups on all four tips for extra security
- Drainage holes in seat to help reduce the build up of water
- Minor assembly required
- 12 months warranty (conditions apply)



Specifications

Seat Heights Adjusts :	*47cm-57cm
Seat Width:	68cm
Seat Depth:	41cm
Overall Width at Base:	74cm
Overall Depth at Base:	57cm
Weight Capacity:	114kgs
Net Weight:	5kgs

*Note:
Seat height measurement refers to the top of the seat.
The maximum clearance on the underside of the bench to clear a bath wall is 52cm.

Instructions

1. To insert the arm support, simply push the support into the guides on the frame until the push buttons snap into position.
2. The backrest can be placed on either side of the seat. Once you have identified the appropriate side, guide the two aluminium support bars through the rings on the underside of the seat. Ensure the push buttons snap into position.
3. Adjust the height of the bench until it is level and at a comfortable height for the user. Ensure that snap buttons are securely locked into position. Offset holes on the extension leg allow more options for levelling the bench on uneven surfaces.
4. To release the suction grip on the large pad, pull on the raised rubber tips to break the seal.



Care and Maintenance

- Check rubber tips for tears or cracks, replace as necessary.
- Check the leg extensions are securely locked in.
- Ensure all screws, nuts, bolts etc are securely tightened.
- Regularly clean with warm soapy water. For tougher stains use a light cream cleanser.

Spare Parts

Replacement tip: RTIP104

Replacement Leg: SBA400 (large suction pad)

Warranty

- Pharmaquip Healthcare offers a one year conditional warranty for domestic use of this product.
- If this product is used for commercial purposes (for example, hire), the warranty period is limited to 6 months.
- Pharmaquip Healthcare warrants its products to be free from defects in material and workmanship.
- Pharmaquip Healthcare will repair or replace any part, provided the product is delivered, pre-paid, to a location authorised by Pharmaquip Healthcare within the prescribed period.
- This warranty does not include parts subject to wear and tear.
- This warranty does not cover damage caused by misuse, neglect, accident or improper use.
- Warranties issued by Pharmaquip Healthcare will be limited to repair or replacement of the defective product.
- Pharmaquip Healthcare will not be liable for any consequential or incidental damages whatsoever.
- This product is intended to be an aid in bathing in the bath or shower. Pharmaquip Healthcare will not be liable for any other improper use.