

## Large Base Quad Cane

Product Code: RCN0102BL Black  
RCN0102BR Bronze  
RCN0102CH Chrome

The Large Base Quad Cane provides additional stability with a large four point cane base, and is suited to those who need more support than a small base quad cane offers.

The cane is height adjustable with a push button (accessible to either a left or right handed person), and has a locking knob for added safety.

The offset handle is covered by a foam hand grip and is available in a Black, Bronze or Chrome finish.

### Features

- Sturdy aluminium construction with lightweight feel
- Double height adjustment button to allow for either left or right handed use
- Constructed with 22mm anodised aluminium
- Easy grip foam handgrip
- Secured with locking knob for added safety
- Non-slip rubber tips (with metal inserts) ensure a secure grip
- No assembly required
- 12 months warranty (conditions apply)

### Specifications

Handle Height: 76cm to 99cm  
Base Size: 21cm wide x 30cm deep  
Weight Capacity: 114kg  
Cane Weight: 1.2kg



Replacement 16mm  
Cane Tips. Our code  
RTIP109



Double button allows the cane  
base to be swivelled around to  
adapt to a left or right handed  
person

## Measuring a Cane

To find the appropriate handle height:

1. Put on your usual walking shoes.
  2. Stand upright with your arms at your side with a normal relaxed bend in your elbow.
  3. The cane needs to be adjusted so the top of the handle equals the position of the wrist joint.
- This is a general measurement guide. Certain users may have specific conditions that will require them to have the cane at a different height. Each individual should always consult with their doctor or therapist to determine proper adjustment and usage.

## Cane Height Adjustment

To adjust the cane to the appropriate handle height:

1. Turn the locking knob counter clockwise to loosen.
2. Slide the extension leg to adjust to your required height.
3. Ensure that the snap button is fully protruding through the height adjustment hole. An audible 'click' will be heard when the push button has securely locked in the hole.
4. Turn the locking knob in a clockwise motion to secure the cane.



## Using a Cane

- Generally, walking sticks are held in the hand opposite to the injured or affected leg, however in some circumstances it may be appropriate to use the cane on the affected side.
- When walking, the cane should be placed forward at the same time as the affected leg is moved forward, the unaffected leg then following.
- Keep the cane away from your feet to avoid tripping.
- Going up stairs: Place the unaffected leg forward first, followed by the cane and affected leg.
- Going down stairs: The affected leg and cane should be lowered first followed by the unaffected leg on the same step.

## Care and Maintenance

The following should be checked periodically:

1. Ensure the locking knob is securely tightened.
2. Check the cane tips for wear and tear.
3. Check the handle to ensure it is attached securely.

## Warranty

- Pharmaquip Healthcare offers a one year conditional warranty on its canes.
- Pharmaquip Healthcare warrants its products to be free from defects in material and workmanship.
- Pharmaquip Healthcare will repair or replace any part, provided the product is delivered, pre-paid, to a location authorised by Pharmaquip Healthcare within the prescribed period.
- This warranty does not include parts subject to wear and tear.
- This warranty does not cover damage by misuse, neglect, accident or improper use.
- Warranties issued by Pharmaquip Healthcare will be limited to repair or replacement of the defective product.
- Pharmaquip Healthcare will not be liable for any consequential or incidental damages whatsoever.
- This product is intended to be an aid for walking, Pharmaquip Healthcare will not be liable for any improper use.