

air imprint memory cushion



Product Code: BE1020

A cushion that memorises your body imprint with the use of an air valve.

By releasing air while sitting on the cushion, once the correct level of comfort is reached, locking the air valve keeps the desired contoured shape.

Reduces risk of developing pressure sores.

- Breathable 3D mesh cover
- Removable, washable cover
- PU foam / Memory foam
- Non-slip base

Specifications

Width:	46cm
Depth:	42cm
Depth inc valve:	44cm
Height:	8cm
Weight:	900g
Weight capacity:	110kg

GENERAL CUSHION CARE

WARNING: Installing a cushion on a wheelchair could affect its centre of gravity and may cause it to tip, potentially resulting in serious injury. Always assess if cushion is suitable to be used on the wheelchair. We recommend anti-tippers be added to the wheelchair for better stability.

TO CLEAN COVER

Remove the cover from the base.

Turn the cover inside out and machine wash in warm water (below 30°C).

Do not bleach.

Do not machine dry after washing – air dry naturally.

When dry, fit cover back over foam base. Ensure the back of the cover matches up with the back of the foam base.

Washing instruction label normally at the base/side of the cover.

TO CLEAN FOAM BASE

Remove the cover – wash separately (see above).

Only wipe foam lightly with a damp cloth and wipe off with a clean cloth. Do not use soap.

Do not submerge in water.

Let the cushion completely air dry before replacing cover.

Do not machine wash or use a dryer.

Do not use iron or hair-dryer on the products – these will damage the material of the products.

