

SAFETY PRECAUTIONS

- Ensure the fold down legs and backrest are locked securely in place.
- Be sure the backrest is in the upright position and all mounting hardware is securely tightened.
- Do not exceed the maximum user weight limit (150kg)
- Do not stand on the shower chair.
- Inspect at regular intervals for wear and damage. Ensure the plastic parts and rubber feet are in good condition; replace rubber feet if damaged.

CLEANING INSTRUCTIONS

The shower chair can be disinfected by spraying it with any standard non-abrasive hard surface cleaner.

1. Gently wipe the disinfectant onto the surface using a soft cloth and let sit for several minutes.
2. Rinse the shower seat to make sure all disinfectant has been removed.
3. Wipe dry before storing.

SPECIFICATIONS

| | |
|-----------------------|----------------|
| Seat Height: | 49 - 58.5cm |
| Seat Width: | 41cm |
| Seat Depth: | 40.5cm |
| Seating Width: | 51cm |
| Seating Depth: | 42cm |
| Backrest Height: | 37cm |
| Armrest Height: | 11.5cm |
| Under Seat Clearance: | 37cm - 47cm |
| Inner Leg Clearance: | 42cm |
| Overall Width: | 66.5cm |
| Overall Depth: | 55cm |
| Weight Capacity: | 150kg |
| Folded Dimensions: | 60 x 50 x 30cm |
| Net Weight: | 5.8kg |

IMPORTANT SAFETY INFORMATION BEFORE USE

It is the user's responsibility to ensure the chair is suitable for use.

- Do not exceed the maximum user weight limit of 150kg.
- Always ensure that the legs are evenly adjusted so the chair is level and stable.
- Inspect at regular intervals for wear and damage. Ensure the plastic parts and rubber feet are in good condition; replace rubber feet if damaged.
- Do not stand on the chair.
- Never leave the user unattended.
- Not recommended for use with individuals who are severely mentally, cognitively, physically or vision impaired.

NOTE: Failure to follow the above noted safety instructions can lead to injuries.

The RITZ range

RTZ-B-200 Deluxe Foldable Commode Shower Chair



Step 1

Remove parts from box and plastic bags.



Step 2

Set the seat upside down on a flat surface.

Loosen the 4 grey plastic screws holding each leg. (circled)




Step 3

(a) Raise legs to upright position.

(b) Align the legs on the seat frame. Insert leg tubes into the seat tubes on each side.

(c) Tighten all four grey plastic screws holding on leg. Make sure the legs latch on each side of the chair, engaging them in the locked upright position.

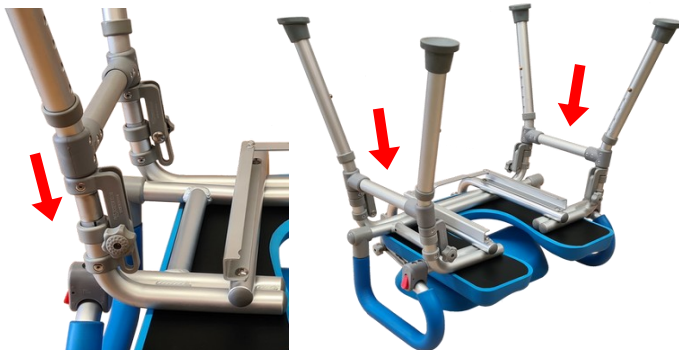


Step 4

(d) Raise the backrest and align it with the lower back support frame.

(e) Insert the backrest into lower back support tubes.

(f) Tighten the two grey plastic screws holding the backrest. Ensure each screw is securely tightened.

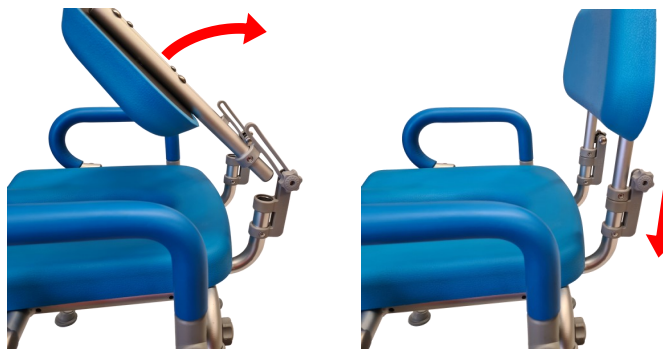

Step 5

Ensure the armrest is locked into armrest socket.

Step 6

Insert the splash guard or pail under the seat.

The shower chair commode is now ready to use.



OPERATION

Adjust the height of the shower chair to the most comfortable position for the client. This is accomplished by pressing the gold coloured button on the leg of the chair and raising or lowering the leg base. Ensure all four legs are aligned to the same height to provide a level base. The rubber feet can be rotated to create a flat contact with the floor.

Push the armrest down until it clicks into a secured position. This will provide extra support and stability for the client during the shower or toileting. To raise the arms push the red button on the side of the arm and swing the arm up.

Insert the desired bucket under the seat from the front for use as a commode or over toilet aid:

Commode

The chair can be used as a commode by utilising the regular pail supplied.

Over Toilet Aid

Insert the splash guard (pail with an open bottom) to assist in directing waste and fluids into the toilet. Position over toilet bowl.

Shower Chair

The chair is also suitable for use as a Shower Chair.

The client should be comfortably seated, centred on the chair. **ALWAYS ENSURE STABILITY**