

RBN104L User Guide

Rotating & Sliding Transfer Bath Bench



PQUIP[®]
PHARMAQUIP HEALTHCARE

PQ HEALTHCARE AUSTRALIA Pty Ltd t/a
PHARMAQUIP HEALTHCARE
1/7 Monte Khoury Drive, Loganholme, Qld 4129
Phone 07 3806 4274 Email sales@pquip.com.au
www.pquip.com.au

PARTS DESCRIPTION

1. Backrest
2. Flip up armrests
3. Armrest latch
4. Safety belt
5. Rotating sliding seat
6. Rotate lock button
7. Slide lock button
8. Height adjustable legs
9. Suction cup feet



ASSEMBLY Step 1

Unbox the shower chair on a level floor.



RBN104L User Guide

ASSEMBLY Step 2

Press the spring button of the leg tube and insert it into the main frame. Repeat for the other side.



ASSEMBLY Step 3

Press the spring button of the leg extension and insert it into the leg tube. Repeat for the other three leg extensions. The two suction tips should be installed on the side that will be placed inside of the bath tub. Once assembly is complete, adjust the lengths accordingly for a level seating surface over the bathtub.



RBN104L User Guide

ASSEMBLY Step 4

Press the spring button of the back brace bar and insert it into the back of the seat.



RBN104L User Guide

ASSEMBLY Step 5

Attach the back end of the armrest into the tube under the seat by pressing in the locking button and insert armrest until it locks in place. Pinch clip on the front and insert into its bracket. Repeat for other side.



RBN104L User Guide

HOW TO SLIDE SEAT

Press the button below the seat (7) and glide along the rail then release.



HOW TO ROTATE SEAT

Pull out the knob under the seat (6), rotate seat then release.



RBN104L User Guide

SPECIFICATIONS

Seat height:	47.7 – 57.8cm
Seat width:	40.5cm
Seat depth:	34.3cm
Seating width:	50cm
Maximum clearance:	44.4cm
Backrest height:	33.7cm
Armrest height:	20.3cm
Seating area:	40.5 x 34.3cm
Overall dimensions:	87.4 x 53 x 91.5cm (at highest setting)
Weight capacity:	136kg
Unit weight:	6.5kg

WARNINGS

To ensure your safety while using this shower chair, all instructions must be followed.

- The maximum load capacity of this shower chair is 136kg.
- Do not use this shower chair if it is obviously damaged or if any parts are missing.
- Ensure all fixed parts are correctly and firmly attached.
- This shower chair should be placed indoors in a dry, ventilated, stable area free of corrosive substances.
- Clean regularly, wiping down with a soft cloth and a mild detergent and/or disinfectant solution.
- Do not use corrosive solutions and/or abrasive cleaners.
- Use caution when bench is set at a higher setting as it may be unstable, there is a greater risk of tipping backwards if the user sits down heavily.