

45cm Transporter Wheelchair



HANDLING AND SAFETY TIPS

Warning: Wheelchairs are heavy items – ensure proper lifting techniques to avoid injury. Ask for assistance whenever possible.

Be sure your weight is within the load capacity of the unit and the wheelchair is suitable for your needs and abilities. Seek the advice of a suitably trained therapist or carer if uncertain.

Before using the wheelchair, always check to ensure product is in good working condition.

Check brakes are in working order.

Ensure both brakes are applied before getting in or out of the wheelchair.

DO NOT stand on the footplates when getting in or out of the wheelchair.

DO NOT overload the wheelchair.

DO NOT hang objects on the attendant handles or backrest – this can alter the centre of gravity and could cause the chair to tip backwards.

Stability

The wheelchair is designed to be safe and stable if the centre of gravity is maintained while going about normal activities. Care must be taken while performing activities that shift the centre of gravity in the chair. Never reach so far that require you to shift in the seat or bend over out of the chair.

Ramps and Inclines

Be sure of your own capabilities, limitations and strength before attempting to go up a ramp. Correct techniques on ramp negotiation should be demonstrated to you by a qualified professional. Avoid changing direction when going down an incline as this could result in instability or tipping over. Seek assistance whenever possible.

Caregivers should navigate downward slopes with the wheelchair turned to face up slope and back the chair downwards gradually.

FEATURES

- Powdercoated aluminium frame
- Plastic handgrips with locking attendant brakes
- Padded seat and backrest with seat belt
- Fold down backrest for easier transportation
- Flip up PU armrests with plastic side guards
- Swing away detachable footrests
- Height adjustable plastic footplates (3 settings)
- PVC front castor
- Solid PU rear mag wheels with brake lock
- Removable anti-tippers
- Conditional warranty applies

SPECIFICATIONS

	PA223
Seat Width:	45cm
Seat Height:	50cm
Seat Depth:	40cm
Backrest Height:	43cm
Armrest Height:	21.5cm
Overall Width:	56cm
Folded Width:	27cm
Overall Length:	95cm
Overall Height:	91cm (backrest folded: 73.5cm)
Front Wheel Size:	19.5 x 2.3cm
Rear Wheel Size:	28.5 x 4.5cm
Weight Capacity:	110kg
Net Weight:	10.80kg

CARE AND MAINTENANCE

- Check condition regularly for wear and tear.
- Do not use if product shows signs of damage or corrosion.
- Always check bolts and nuts are securely fastened.
- Clean using a slightly damp cloth and wipe dry.
- Lubricate moving parts. We recommend a silicone based lubricant which will also assist in water / moisture displacement and can help reduce rust / corrosion.
- It is recommended that the wheelchair be serviced at least once a year by a mobility aid specialist.





- | | |
|--------------------------|----------------------------------|
| 1. Attendant hand grip | 6. Hand brake |
| 2. Attendant hand brake | 7. Swing away leg rests |
| 3. Padded nylon backrest | 8. 12" PU rear wheel |
| 4. Flip-up arm rests | 9. 8" Front castor |
| 5. Padded nylon seat | 10. Height adjustable footplates |

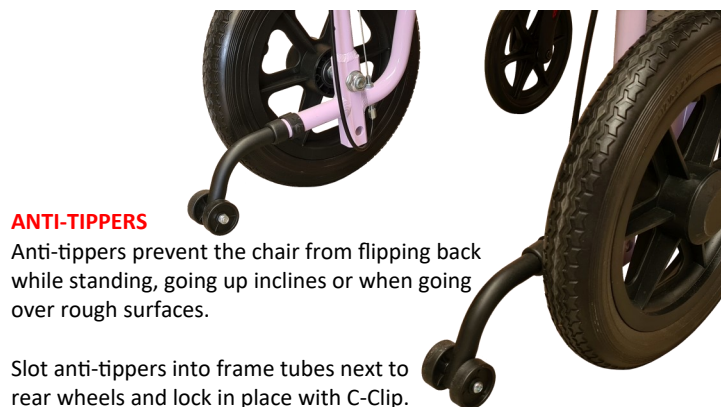


Figure A. Anti-tippers

UNFOLDING the Wheelchair (Figure B)

Hold the attendant handgrips with both hands to separate the frame slightly. Then push down on the sides of the seat (5a) until frame is fully extended.

FOLDING the Wheelchair (Figure B)

Step 1: Turn up both footplates (10).
Step 2: Fold the wheelchair by pulling up the handles (5a) on the sides of the seat.



Figure B. Unfolding / Folding

BRAKE SYSTEM

Important: Ensure both brakes are applied before getting in or out of the wheelchair.

Attendant hand brake (2)

(Figure C): Squeeze (2a) upward to apply brakes. Brakes lock when (2b) clicks in. Squeeze (2b) upward to release brake lock.



Figure C. Attendant Brake

Hand brake (6)

(Figure D): Push brake handle (6) forward to engage park brake. Pull back (6) to release.

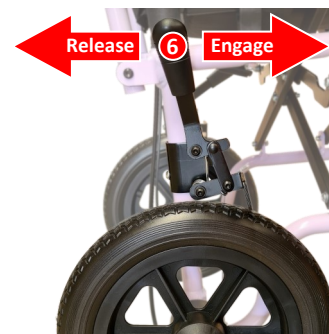


Figure D. Hand Brake

Note: When not in use, DO NOT leave hand brake (6) engaged as this may cause permanent indents on the tyre.

FLIP UP ARMRESTS (Figure E)

a To Open

Depress release lever (4a).
Rotate arm to the back of wheelchair.

b To Fasten

To return armrest to down position, push armpad down until release latch (4a) locks firmly into place (4b).



Figure E. Flip Up Armrests

FOOTRESTS (Figure F)

Installation

- Place swing-away pivot pin into the locating hole on top of the hanger mount with the footrest facing outward from frame (7a).
- Rotate the footrest inward until latch plate locks (7b) into place on locking bolt.

Removal

- To remove footrest, press quick-release lever (7b).
- Rotate footrest outward and lift.



Figure F. Swing Away Footrests