

PAB802 HEAVY DUTY GUTTER ARM WALKER



1. Length adjustable handles
2. Handbrake lever
3. Gutter arm support
4. Handle length adjustment
5. Gutter height adjustment
6. Removable back rest
7. Lift up seat
8. Storage bag
9. Folding bracket
10. Rear wheel with brake
11. Front wheel

HOW TO USE

UNFOLDING THE ROLLATOR

Spread apart the front and rear legs Push down on the folding bracket (9) on each side to full extension.

FOLDING THE ROLLATOR

Lift the seat open, using the strap handle inside the storage bag, pull up from handle strap inside bag and push the rollator frame together to fold


Caution: Be careful that fingers or any objects do not get caught between the hinge arms (9) when closing

GUTTER ARMS – INSTALL / ADJUST HEIGHT

Insert gutter arm bars into frame tube (5)
Adjust position to required height; locate nearest hole and insert bolt. Ensure the bolt head recesses into the 6 side hole in frame; slide washer on with the curved side facing the frame; then screw on knob and tighten. Repeat for other side.

ADJUSTING ARM LENGTH

Insert handles into gutter arm (4) and adjust to desired length and position. Tighten nut and flip lever to lock in place.

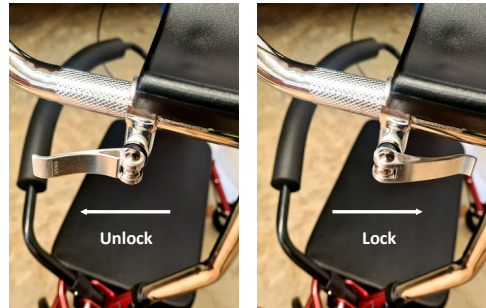

Locking lever for handle bar

GENERAL CARE & MAINTENANCE

- Seek the advice of a suitably trained therapist on proper use.
- Ensure your weight is within the weight capacity of the unit and the product is suitable for your needs and abilities.
- Before use, always check product is in good working condition. Check regularly for wear and tear.

Care must be taken to ensure stability:

- All castors must be in contact with floor.
- Avoid tilting or leaning over too far.
- DO NOT use as a wheelchair or to transport someone.
- Lock handbrakes when sitting, while seated and getting up.
- Backrest must be attached to walker prior to use.

DO NOT use if rollator shows signs of damage or corrosion.

- Regular service is advised
- Periodically check that bolts and nuts are securely fastened.
- Lubricate moving parts.
- Clean using a slightly damp cloth and wipe dry.
- Do not submerge the product in water and make sure to dry product completely before storing. Store in a dry place.

SPECIFICATIONS

Gutter arm height:	102.5cm – 111cm
Seat height:	54cm
Seat width:	46cm
Seating width	56cm
Overall depth	66cm (front Castor Inward)
Overall width:	74cm
Overall height:	127cm
Folded width:	50.5cm
Width between gutter arms:	49cm
Wheel Size:	19 x 4.5cm PVC
Weight Capacity:	180kg
Net Weight:	12kg

