


The assembled product is shown in the right figure. A pull rod can be seen below the right side of the seat plate. When the pull rod is pulled, the direction of the seat plate can be rotated and adjusted.

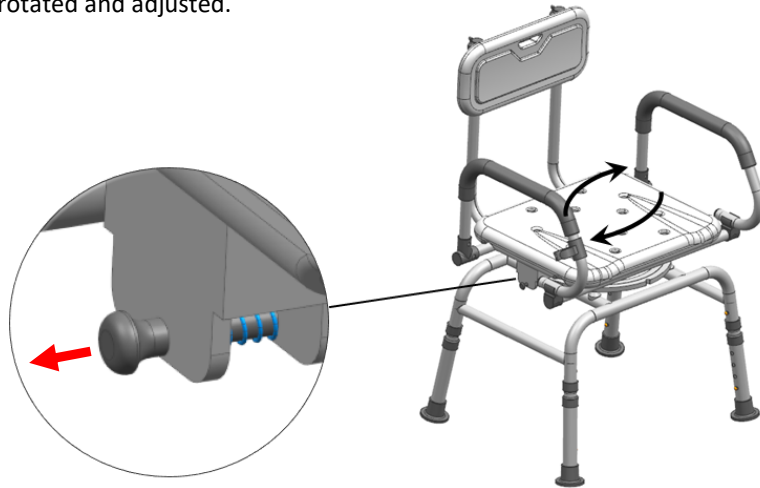

## WARNING:

- Before use, always check product is in good working condition.
- Seek advice of a suitably qualified therapist.
- Ensure the user's weight is within the weight capacity of the unit.
- Care must be taken to ensure stability.
- Avoid tilting or leaning over too far.
- Periodically check all hardware and the height adjustment buttons are secure.
- Stay away from open flames.

## Precautions before assembly:

- Prepare a clean and dry mounting area.
- Verify that the product is not damaged.
- Check that none of the pieces are missing.

## Maintenance:

- Clean with a soft fabric.
- Check the seating regularly.

## Storage conditions:

- Store product completely dry, in a clean and dry place to avoid the risks of mold.
- Caution with the risks of pinching or cuts during assembly.



The manufacturer declines all responsibility of damages due to a incorrect installation.



**RTZ-B103 Swivel Shower Chair**


When unpacking this product, check to make sure the parts listed below are included\* and carefully inspect for any damage that may have occurred during transit. Do not attempt to assemble or use the product if any part is missing or damaged.

*\*Images are for illustrative purposes only and may vary slightly from actual product.*

No.	PICTURE	PART	QTY	No.	PICTURE	PART	QTY
1		Seat frame	1	5		Curved washer	4
2		Chair leg	4	6		Backrest tube	2
3		Backrest	1	7		Armrest	2
4		Knob screw	4	8		Shower buckle	1


1. Insert the 4 chair legs (2) into each of the tubes located on the underside of the seat frame (1). Ensure that the push button on each chair leg clicks out fully and the legs are secure.

2. Set the chair to an appropriate seat height. The legs are adjustable to four levels. Ensure that the push button on each chair leg clicks out fully and the legs are secure.

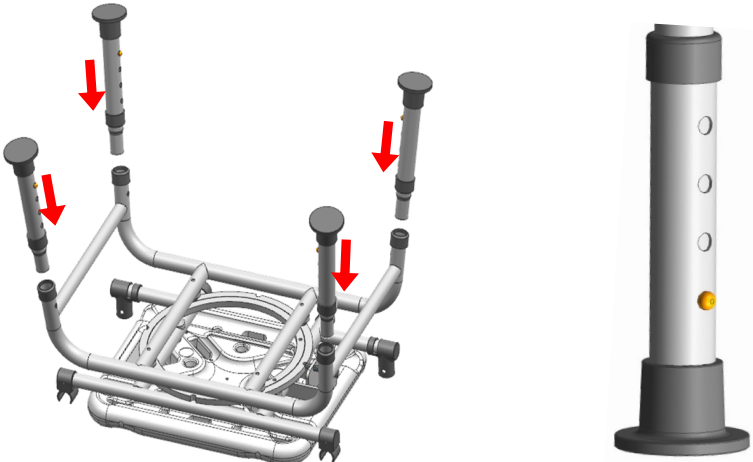

3. Attach the two armrests (7) to each side of the seat frame. Ensure that the push button on each clicks out fully. Lock the front section into the seat guide.

Note: To raise the armrest, pull it up to release the arm from its guide and rotate it back out of the way (it may require a little force to clear the support guide (A)).

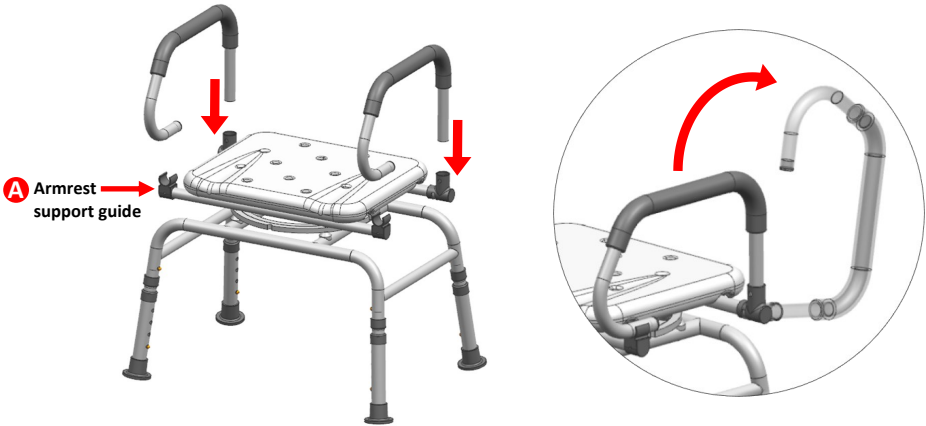

4. Insert the assembled backrest (B) into the corresponding pipe opening of the main frame in the direction shown in the diagram (ensure that the marble of each backrest pipe pops out fully).

The position of the shower head buckle (8) can be adjusted for better convenience.

