

Wheelchair Anti-Tippers

Description: **Wheelchair Anti-Tippers (A)**

Part No: SWH921
Suit Model: PA140, PA145, PA160, PA190,
PA195

- These Anti-Tippers, with castors, are suited for our PA Series of chrome wheelchairs.
- They are designed to slide over a 22mm rear step tube and are held in place by a brass push button.
- The PA Series of chrome wheelchairs have two pre-drilled holes in the step tube frame to attach the anti-tippers.
- Sold as a pair.



To fit the Anti-Tipper



Remove the step tube end cap to expose the two, pre-drilled holes in the frame.



Insert the double button spring into the tube until the brass buttons snap into position in the holes.



Depress the brass button and slide on the anti-tipper over the step tube until it locks in place.



Wheelchair Anti-Tippers

Description: **Wheelchair Anti-Tippers (B)**

Part No: SWH925
Suit Model: PA200
Tube Diameter: 18mm
Overall Length: 25cm
Height Adjusts: 12cm to 17cm



To fit the Anti-Tippers

- These Anti-Tippers both work by sliding the tube into the wheelchair frame, then securing it in place using a U shaped clip.
- The 2nd U shaped clip allows for height adjustment
- The SWH925 model is both height and length adjustable, while the SWH927 is height adjustable only.
- Sold as a pair



Description: **Wheelchair Anti-Tippers (C)**

Part No: SWH927
Suit Model: PA210
Tube Diameter: 21mm
Overall Length: 25cm
Height Adjusts: 12cm to 15cm

